phyteneo Occusept

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C-400-192-16-R0 z 1. 3. 2017

GB Occusept:

Carefully read the Instructions for Use before using. Keep the Instructions for Use for possible future reference.

Intended Purpose of Use: Occusept is a medical device for the supportive treatment of human eye diseases, for disorders in the creation of tears of a various etiology, dry eye syndrome of a various etiology, due to the use of certain drugs or environmental conditions (high temperature, wind, etc.), for soothing the eyes in acute or chronic conjunctivitis, of a non-infectious origin, for irritated eyes due to external conditions (sun, dust, smoke, etc.), due to wearing contact lenses or during long-term eye strain. The solution may be used for rinsing the eyes, when they come into contact with irritating agents and for removing foreign substances.

Ingredients: Hypromellosum 6 mg, acidum boricum 8.5 mg, natrii tetraboras decahydricus 2 mg in 1 ml. Aqua purificata, Euphrasia officinalis extractum fluidum, natrii chloridum, zinci sulfuri pentahydricus, phosphatidylcholine, phosphatidylethanolamine, tocoferoli alfa acetat, caprylic / Capric triglyceride, polysorbatum 80, dinatrii edetas dihydricus, natrii hydroxidum.

Characteristics: Occusept is an ophthalmologic, eye drops. It is a low viscosity, isotonic solution with Euphrasia officinalis (Eyebright) extracts, which is traditionally used for inflammation of the conjunctiva accompanied by redness and burning of the eyes, during increased tearing, swelling and redness of the eyelids or eyelid margins. The solution contains lipid components - LipoZoom™, fractions of purified soybean lecithin phosphatidylcholine, phosphatidylethanolamine, which occurs naturally in tears and improves the coverage of the eye by a protective film (it prolongs the period when the film is not interrupted by creating a lipid layer, which prevents evaporation and interruption of the continuous layer of tear film), which significantly improves the patient's comfort during the malfunction of tear creation.

Indications: The product is used as a substitute for tears during the malfunction of tear creation, for creating a protective film when exposed to undesired external conditions, during acute or chronic inflammation of conjunctiva, eye irritation, fatigue following long period of eye strain and for rinsing of the eyes.

Contraindications: Hypersensitivity to any of the agents stated in the product's ingredients.

Interactions: The product cannot be used in combination with eye drops, which contain metal salts.

Undesired Effects: Unknown. Consult your eye doctor upon the occurrence of unusual reactions.

Dosing and Method of Application:

The product is dosed individually. The usual dose is 2 to 3 times per day, 1 to 2 drops into the conjunctiva. The product can also be used long-term, as needed.

INSTRUCTIONS FOR USING THE DROP LOCK:

- Turn the cap counter clockwise to tear off the security strip.
- Remove the cap and turn the bottle upside down.
- Using the finger tips of one hand, pull the lower eyelid down and using the other hand, lightly press the prescribed amount of drops into the eye or conjunctiva, which is revealed once pulling the lower eyelid down.
- When applying the eye drops, prevent the eye dropper from direct contact with the eyes or the skin and do not touch the tip of the eye dropper to avoid contamination of the solution.
- After use, always put the cap back on and close tightly.
- If used for rinsing of the eye, press the bottle several times until the fill the eye cup up to the mark. Apply the eye cup to the effected eye then tilt your head so that the eye is submerged in the liquid. Move the eye so that the entire surface of the eyeball is wet. The recommended time for rinsing is 2-5 minutes.
- If you wear contact lenses, apply Occusept in the case of inflammation, eye irritation and rinsing of the eye, only once you are not wearing the contact lenses. Contact lenses can be put back in 30 minutes after using the product.
- When wearing contact lenses, use Occusept for the sole purpose of moistening the surface of the eye.

According to the information available, Occusept can also be used by children, pregnant and nursing women.

Warning:

- If you are using other eye medication, consult your eye doctor about the suitability of parallel use. If your condition does not improve after 7 days or if it even worsens, consult an eye doctor.
- Soft contact lenses should not be worn if inflammation or eye irritation occurs when using eye drops or during rinsing of the eyes. Soft contact lenses can be worn 30 minutes after using the product.
- The content of one bottle should only be used by one person. This decreases the transfer of possible infection.
- If accidentally consumed, drink a larger amount of water and contact your physician.
- Children younger than 3 years of age can use the product only following consultation with a specialist.

Storage:

- Keep out of the reach of children. Keep in its original package. Close tightly after opening and keep it stored in a vertical position.

Warning:

If you notice any undesired effects, consult your physician. Do not use the product if it is turbid or if it has changed color. The product cannot be used once the expiration date stated on the package has elapsed, even if the bottle had never been opened. Once the bottle has been opened, the product can only be used for a period of 30 days.

Packaging: Occusept packaging contains 2 x 20 ml solution filled in a white, plastic bottle with a cap. The packaging also includes a small plastic eye cup for rinsing of the eye.

Liquidation: Return the unused product back to the pharmacy. Throw the clean package into the the related recycle bin.

Neofyt, Ltd., Pražská 442, 281 67 Stříbrná Skalice, Czech Republic
Tel.: (+420) 311 230 754, E-mail: phyteneo@phyteneo.cz

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